



TRACE EYE-D[®]



Fentanyl



**DRUG
DETECTION
WIPES**

**MADE
IN THE USA**

Detects: Detects All Opioids. Also
Detects: Fentanyl, Carfentanil, Heroin,
Morphine, Oxycodone, Hydrocodone,
Xylazine, Metonitazene, Tramadol,
Benzyl Fentanyl, Meperidine.

TRACEEYED.COM



DRUG DETECTION WIPES



Positive test result color

Directions | #OWLmethod

▶ **OPEN** pouch and remove wipe.

Do not unfold towelette before or after use - doing so may result in difficulty observing the results.

▶ **WIPE** it across the surface to be tested.

For best results, sample small amounts of the substance on the outer edge of the wipe.

▶ **LOOK** to see if the wipe has changed color.

Most results are instant, however, some may take up to 30 seconds for a color reaction to develop.

- Negative Result: NO color change
- Positive Result: Magenta